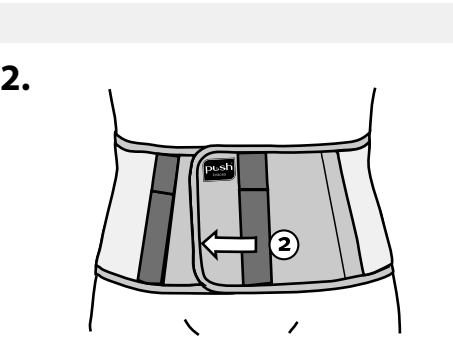
A.

B.

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PUSH CARE BACK BRACE

INTRODUCTION

Push care products provide effective solutions when your freedom of movement is restricted by an injury or disorder.

Your new back brace is a Push care product, characterised by a high level of effectiveness, comfort and quality. The Push care back brace provides pain relief by supporting the lumbar vertebral column.

Push care products are innovative and are manufactured for you with great care. This ensures maximum security and carefree use.

Read the product information carefully before you use the Push care back brace. If you have any questions, please consult your reseller or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

Your Push care back brace offer relief for lower-back complaints by evenly distributed compression. A soft, comfortable pad applies additional pressure locally on the lower back. This pad can be moved.

Because of its design and the materials used, the brace is comfortable to wear during standing or sitting activities, both at home and at work. Its anatomical fit is suitable for both men and women, and provides appropriate position retention. The close body-fit of the brace renders it invisible underneath your clothes. Compression can be adjusted easily to suit your preferences. The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the back part feel soft and keeps the skin dry. The Push care back brace is easy to apply and can be washed without any problem.

INDICATIONS

- Non-specific lower-back complaints
- الشكوى والآلم الظهر غير المحددة
- For prevention in case of back-straining activities
- Pseudoradicular complaints
- Mild lumbar arthrosis

There are no known disorders where the brace cannot or should not be worn. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING

The back brace is available in 6 sizes. The size is determined by the circumference of the waist. The table on the packaging shows the correct brace size for your waist.

A. Wear the back brace against the skin or over your undergarments. Please ensure that the brace is positioned in the centre of the vertebral column at the level of the lower back. The bottom of the brace covers the hips.

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