ENGLISH

Medical Device

01/2022

1 1039A-40 1 / 3

Made in the EU

Designed in the Netherlands

www.push.eu


The Netherlands

6199 AB Maastricht Airport

Nea International bv

1 40 1

REF

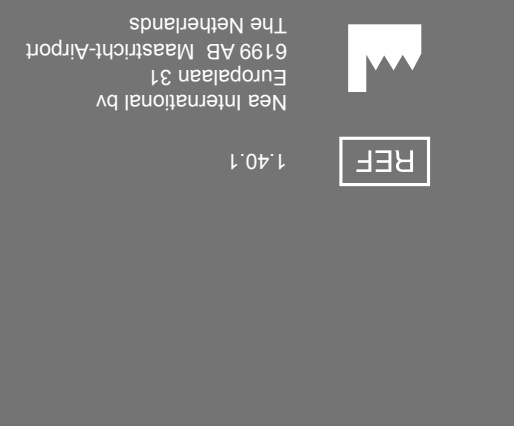## CARE

### BACK BRACE

## PRODUCT INFORMATION

EN • AR • PY • KRN • SL

EL • YKP • CHN • FR • RO

• LT • LV • TR •

### ENGLISH

### PUSH CARE BACK BRACE

#### INTRODUCTION

Push care products provide effective solutions when your freedom of movement is restricted by an injury or disorder.

Your new back brace is a Push care product, characterised by a high level of effectiveness, comfort and quality. The Push care back brace provides pain relief by supporting the lumbar vertebral column.

Push care products are innovative and are manufactured for you with great care. This ensures maximum security and carefree use.

Read the product information carefully before you use the Push care back brace. If you have any questions, please consult your retailer or send an e-mail to [info@push.eu](mailto:info@push.eu).

#### PRODUCT CHARACTERISTICS

Your Push care back brace offer relief for lower-back complaints by evenly distributed compression. A soft, comfortable pad applies additional pressure locally on the lower back. This pad can be moved.

Because of its design and the materials used, the brace is comfortable to wear during standing or sitting activities, both at home and at work. Its anatomical fit is suitable for both men and women, and provides appropriate position retention. The close body-fit of the brace renders it invisible underneath your clothes. Compression can be adjusted easily to suit your preferences. The brace is made of Sympress™, a high-quality, comfortable material. The use of microfibres makes the back part feel soft and keeps the skin dry. The Push care back brace is easy to apply and can be washed without any problem.

#### INDICATIONS

- Non-specific lower-back complaints
- الشكوى وآلام أسفل الظهر غير المحددة
- For prevention in case of back-straining activities
- Pseudoradicular complaints
- Mild lumbar arthrosis

There are no known disorders where the brace cannot or should not be worn. Avoid applying the brace too tightly at first.

Pain is always a warning sign. If the pain persists or symptoms worsen, it is advisable to consult your doctor or therapist.

#### DETERMINING SIZE / FITTING

The back brace is available in 6 sizes. The size is determined by the circumference of the waist. The table on the packaging shows the correct brace size for your waist.

- Wear the back brace against the skin or over your undergarments. Please ensure that the brace is positioned in the centre of the vertebral column at the level of the lower back. The bottom of the brace covers the hips.
- Hold the brace at the bottom and close the hook and loop fastener. In doing so, tension the elastic material. After fitting, the brace should apply a comfortable pressure. If the pressure experienced is too tight, the brace should be slackened. You may also use the grips to close the brace.

#### WASHING INSTRUCTIONS

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30 °C, on a delicate fabric cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (**not** near a heating source or in a tumble drier).

#### IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before purchasing, to check its size. Your retailer’s professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace before each use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the appropriate regulatory body in the user’s country of residence.

Push® is a registered trademark of Nea International bv

For details of the distributor in your country, we refer to [www.push.eu/distributors](http://www.push.eu/distributors)

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