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#### PUSH MED ELBOW BRACE EPI

#### INTRODUCTION

Push med products provide effective solutions for the treatment and prevention of injuries of the locomotor apparatus.

Your new elbow brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med elbow brace Epi improves functioning and provides pain relief. This increases physical strength, Mechanical support, adjustable pressure and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push med elbow brace Epi. If you have any questions, please consult your retailer or send an e-mail to **info@push.eu**.

#### PRODUCT CHARACTERISTICS

Your Push med elbow brace Epi offers local pressure on the extensors (tennis elbow) or the flexors (golfer's elbow) of the wrist. This relieves the strain where these muscles come in contact with the elbow joint. The brace can easily be fitted and closed around the lower arm with one hand. The bracing element around the arm ensures proper distribution of pressure. The padding consists of a skin-compatible and elastic polymer. A proper anatomic fit and the material of the padding ensures that the brace remains in place during movement.

The brace is made of Sympress™, a high-quality and comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push med elbow brace Epi can be washed normally.

#### INDICATIONS

- Tennis elbow (lateral epicondylitis)
- Golfer's elbow (medial epicondylitis)

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly at first.

Pain is always a warning sign. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

#### DETERMINING SIZE / FITTING

The elbow brace Epi is available in 1 size. The measurements range from 20 to 35 centimetres.

Make sure that the brace's pressure point (at the level of the lgo) is placed a few centimetres beside the painful insertion of the muscle towards the hand.

Tighten the adjustable strap to desired tension level and close it.

#### WASHING INSTRUCTIONS

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

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