PUSH ORTHO ANKLE BRACE AEQUI JUNIOR

Please read this product information carefully before using the Push Aequi Junior.

INTRODUCTION

Push products provide effective solutions for a variety of disorders of the locomotor apparatus. The products can also be used as a preventive measure after previous injuries of the joints. The Push ortho ankle brace Aequi Junior is a Push Braces product from the so-called ortho category. Push ortho products are innovative and are carefully designed to provide the highest level of comfort, maximum support and effective protection. The Push ortho Aequi Junior ankle brace was especially designed for children with unstable ankles.

The Aequi Junior is characterized by a high degree of support, effectiveness, comfort and quality. The product provides enhanced stability and helps control pain. A major advantage of wearing this brace is that it increases load bearing capacity. This restores the ankle joint's full functionality more quickly, and most importantly, more safely.

PRODUCT CHARACTERISTICS

The Aequi Junior is an effective ankle brace for children (or adults with a small instep) aged between 6 and 12, with an instep of 23-27 cm (shoe sizes 12-13 and adult shoe sizes 1-4). UK: instep 9-10½" (children shoe sizes 12-13 and adult shoe sizes 1-4).

This ankle brace is suitable for both the left and right ankle and is available in a single size.

The brace offers a great balance between effective mechanical ankle support and optimal wearer comfort. It consists of a sturdy foam shell and a set of elastic and non-elastic straps.

The Aequi Junior provides the required support of the ligaments. Elastic straps also provide pressure around the ankle joint.

The inversion-eversion (inward-outward movement of the foot) is limited, while plantar-facial flexion (bending and stretching) is allowed, adequate freedom.

This means that the foot is felt during normal walking and running.

The numbers on the straps allow for an easy fitting of the brace.

Parents and carers are capable of fitting the brace themselves.

For proper functioning of the brace, however, it is important that parents or carers initially observe and provide any help needed and inspect whether the Aequi Junior has been fitted correctly.

Maximum stabilisation is achieved when the brace is worn inside a closed (lace-up) shoe. When children play indoors, the Aequi Junior can also be worn without a shoe, while retaining the function of stabilisation.

INDICATIONS

- In the case of early functional therapy for injuries to the medial or lateral ankle ligaments
- Following treatment of ankle fractures that have been operated on or conservatively managed
- In the case of (residual) instability (increased tendency to sprain)
- To prevent (recurrence of) ankle ligament injuries

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly at first.

Pain is always a warning sign. If the pain persists or the symptoms worsen, it is advisable to consult your doctor or therapist.

Always follow the advice of your doctor or therapist.

FITTING

The hook and loop fastenings are marked with symbols. These are to help you fasten the hook and loop material in the correct sequence. First on then on .

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