PUSH CARE WRIST BRACE

INTRODUCTION

Push care products provide solutions when your freedom of movement is restricted by an injury or disorder.

Your new wrist brace is a Push care product, characterised by a high level of effectiveness, comfort and quality. The Push care wrist brace offers pain relief and a greater sense of stability by means of adjustable compression.

Push care products are innovative and are manufactured for you with great care. This ensures maximum security and carefree use.

Read the product information carefully before you use the Push care wrist brace. If you have any questions, please consult your retailer or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

Your Push care wrist brace provides compression around the wrist, improving the sense of stability of the joint. This compression can be adjusted as needed. The wrist joint is not immobilised, leaving you sufficient freedom of movement to carry out activities. The palm of the hand is left free, to allow a proper grip.

So-called "comfort zones" of padded material have been created around vulnerable areas of the wrist. A proper anatomic fit contributes to the brace remaining in position during movement.

The brace is made of Sympress™, a high-quality, comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push care wrist brace is easy to apply and can be washed without any problem.

INDICATIONS

- Slight wrist strain
- Slight distortion
- Slight residual instability
- Mild arthrosis
- Slight rheumatoid complaints


There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly at first. Pain is always a warning sign. If pain persists or symptoms worsen, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING

The wrist brace is available in 4 sizes. Measure the circumference of the wrist as shown in the illustration on the packaging. The chart on the packaging shows the correct brace size for your wrist.

- A The correct fitting sequence is determined by the increasing molecule symbols on the wrist brace. Slide the base around the hand until it reaches the thumb. Close the first fastening around the wrist.
- B Wrap the long strap around the wrist to apply the required compression to the wrist.
- C Close the hook and loop fastener.

WASHING INSTRUCTIONS

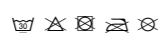To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30 °C, on a delicate fabric cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Push braces were designed to be worn on undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Try on the brace before purchasing, to check its size. Your retailer's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace before each use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the appropriate regulatory body in the user's country of residence.

Push® is a registered trademark of Nea International bv

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