**PUSH CARE ANKLE BRACE**

**INTRODUCTION**

Push care products provide solutions when your freedom of movement is restricted by an injury or disorder.

Your new ankle brace is a Push care product, characterised by a high level of effectiveness, comfort and quality. The Push care ankle brace helps relieve pain and creates a sense of greater stability by means of adjustable pressure.

Push care products are innovative and are manufactured for you with great care. This ensures maximum security and carefree use.

Read the product information carefully before you use the Push care ankle brace. If you have any questions, please consult your retailer or send an e-mail to [info@push.eu](mailto:info@push.eu).

**PRODUCT CHARACTERISTICS**

Your Push care ankle brace provides compression around the ankle, improving the sense of stability of the joint. This compression can be adjusted as needed.

The ankle brace has a perfect fit and can be worn in practically any shoe. So-called comfort zones of padded material have been created around vulnerable areas of the ankle.

The brace is made of Sympress™, a high-quality, comfortable material. The use of microfibrres makes the inside feel soft and keeps the skin dry. The Push care ankle brace is easy to apply and can be washed without any problem.

**INDICATIONS**

- Slight distortion
- Slight capsule irritation
- Slight residual instability
- Arthrosis without instability

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia.

Avoid applying the brace too tightly at first.

Pain is always a warning sign. If pain persists or symptoms worsen, it is advisable to consult your doctor or therapist.

**DETERMINING SIZE / FITTING**

The ankle brace is available in 5 sizes. Position the foot at an angle of 90° in relation to the lower leg. Then measure the circumference around the heel and the top of the foot as shown in the illustration on the packaging. The table on the packaging shows the correct brace size for your foot.

A. Slide the base around your foot, making sure it fits snugly. Keep the foot at an angle of 90° in relation to the lower leg. Then close the elastic fastening at the front.

B. Wrap the long elastic strap around the front part of the foot.

C. Close the long elastic strap on the inside of the foot over the short elastic strap just above the ankle bone.

**WASHING INSTRUCTIONS**

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30 °C, on a delicate fabric cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

**IMPORTANT**

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Try on the brace before purchasing, to check its size. Your retailer's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace before each use for signs of wear or ageing of parts or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the appropriate regulatory body in the user's country of residence.

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