PUSH CARE KNEE BRACE

INTRODUCTION

Push care products provide effective solutions when your freedom of movement is restricted by an injury or disorder.

Your new knee brace is a Push care product, characterised by a high level of effectiveness, comfort and quality. The Push care knee brace provides pain relief and a sense of additional stability by means of adjustable compression.

Push care products are innovative and are manufactured for you with great care. This ensures maximum security and carefree use.

Read this product information carefully before you use the Push care knee brace.

If you have any questions, please consult your retailer. Alternatively, ask your question through info@push.eu.

PRODUCT CHARACTERISTICS

The Push care knee brace provides compression around the knee joint, improving the sense of stability of the joint.

The brace supports the patella on both sides. The compression and patella support can be adjusted as needed. The horseshoe-shaped patella element stabilises and relieves the patellofemoral joint. The open construction of the brace's upper part allows the brace to be fitted easily. A proper anatomic fit and silicone lining contribute to the brace remaining in position during waking.

The brace is made of Sympress™, a high-quality, comfortable material.

The microfiber used makes the inside soft to the touch and keeps the skin dry.

The Push care knee brace is easy to apply and can be washed without any problem.

INDICATIONS

- Osteoarthritis of the knee joint
- Patellofemoral osteoarthritis
- Retropatellar pain complaints
- Patellofemorales Schmerzsyndrom
- Patellofemorale pain syndrome
- Patella instability
- Rheumatisches Beschwerden
- Rheumatoid complaints
- Residual functional instability

There are no known disorders where the brace cannot or should not be worn.

In the case of poor blood circulation in the limbs, a brace may cause hyperaemia.

Avoid applying the brace too tightly at first, in particular at the back of the knee.

Pain is always a warning sign. If the pain persists or symptoms worsen, it is advisable to consult your doctor or therapist.

FITTING / SETTING

The knee brace is available in 5 sizes. Position the upper leg at an angle of 90° in relation to the lower leg. Then measure the circumference at the narrowest section underneath the knee joint. The table on the packaging shows the correct brace size for your knee circumference.

Loosen all hook and loop fasteners to facilitate fitting.

A. Pull the brace over the knee joint, until the preformed patella element encloses the knee. Secure the brace by closing the hook and loop fasteners 1 and 2.

B. Close the straps 3 and 4 diagonally across the upper part. Adjust to the most comfortable pressure, by tightening or loosening the straps.

CARE AND MAINTENANCE

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30°C, on a delicate fabric cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used; after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Try on the brace before purchasing, to check its size. Your retailer's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace before each use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the appropriate regulatory body in the user's country of residence.

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