## PUSH CARE NECK BRACE

## INTRODUCTION

Push care products provide effective solutions when your freedom of movement is restricted by an injury or disorder.

Your new neck brace is a Push care product, characterised by a high level of effectiveness, comfort and quality. The Push care neck brace provides pain relief by stabilising the cervical vertebrae. Mechanical support and good anatomic fit constitute the basis of its performance.

Push care products are innovative and are manufactured for you with great care. This ensures maximum security and carefree use.

Read the product information carefully before you use the Push care neck brace. If you have any questions, please consult your retailer or send an e-mail to [info@push.eu](mailto:info@push.eu).

## PRODUCT CHARACTERISTICS

Your Push care Neck Brace provides good support to the cervical spine due to the anatomic shape and the use of semi-rigid foam. The brace is made of Sympress™, a high-quality, comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push care neck brace is easy to apply and can be washed without any problem.

## INDICATIONS

- Complaints caused by whiplash
- Acute neck injury
- Mild cervical arthrosis

There are no known disorders where the brace cannot or should not be worn. Avoid applying the brace too lightly at first.


Pain is always a warning sign. If pain persists or symptoms worsen, it is advisable to consult your doctor or therapist.

## DETERMINING SIZE / FITTING

The neck brace is available in 2 circumferences and 2 heights. Hold the head straight and measure the circumference of the neck from the back over the Adam's apple. The table on the packaging shows the correct brace size for your neck circumference. The required height will be determined during fitting. The head is properly supported when it is in a neutral position.

- Hold the brace with both hands in front of you, so that the inside is visible. The point at the centre of the brace should be pointing down.
- Then close the hook and loop fastener at the back of the neck.

## WASHING INSTRUCTIONS

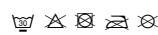To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

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Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30 °C, on a delicate fabric cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

## IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Try on the brace before purchasing, to check its size. Your retailer's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace before each use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the appropriate regulatory body in the user's country of residence.

Push® is a registered trademark of Nea International bv

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