PUSH MED WRIST BRACE

INTRODUCTION

Push med products provide effective solutions for the treatment and prevention of injuries of the locomotor apparatus.

Your new wrist brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med wrist brace improves stability and provides pain relief. This increases physical strength. Mechanical support, adjustable compression and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push med wrist brace. If you have any questions, please consult your retailer or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

Your Push med wrist brace offers a high level of support, protection and rest of the wrist, without restricting the functionality of the hand. The anatomically shaped stiffeners limit primary and dorsal flexion without obstructing them completely. The level of compression can be optimised by an easy-to-apply elastic strap. The brace is easily fitted with one hand around the lower arm.

So-called comfort zones of padded material have been created around vulnerable areas of the wrist. A proper anatomic fit contributes to the brace remaining in position during movements.

The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push med wrist brace is washed normally.

INDICATIONS

- Anspannungsbeschwerden im Handgelenk
- Strain complaints in the wrist
- Instability
- Moderate to serious distortion
- RSI
- Strain complaints in the wrist extensors
- Handgelenkbrüche ohne Dislokationen
- Wrist fracture without dislocations
- Follow-up treatment of wrist fractures
- Arthrosis

There are no known disorders with which the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly at first.


Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING

The wrist brace is available in 4 sizes. Measure the circumference of the wrist, as shown in the illustration on the packaging. The table on the packaging shows the correct brace size for your wrist size.

- The correct fitting sequence is determined by the increasing molecule symbols on the hook and loop fasteners.
 - Slide the base around the hand until it reaches the thumb.
 - Close the brace around the forearm.
- Use the elastic strap to set the correct measurement range around the hand.
- Wrap the strap around the wrist once to apply the required compression to the wrist and close it at the top.

WASHING INSTRUCTIONS

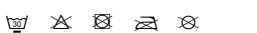To promote hygiene and to maximise the lifespan of the brace it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30 °C, on a delicate fabric cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Try on the brace before purchasing, to check its size. Your retailer's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace before each use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the appropriate regulatory body in the user's country of residence.

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