PUSH MED WRIST BRACE SPLINT

INTRODUCTION

Push med products provide effective solutions in the case of a disorder or reduced functioning of the locomotor apparatus.

Your new wrist brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push Splint improves stability and provides pain relief. This increases physical strength. Mechanical support, adjustable compression and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push med wrist brace splint. If you have any questions, please consult your retailer or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

Your Push med wrist brace splint offers a high level of support, protection and relaxation of the wrist without restricting the functionality of the hand. The brace is easily fitted with one hand around the forearm, the wrist, and the hand. A circular, non-elastic connection ensures that the adjustable splint tightly fits the anatomy of the wrist. This restricts palmary flexion and places the hand in a functional position.

So-called "comfort zones" of padded material have been created around vulnerable areas of the wrist. A proper anatomic fit contributes to the brace remaining in position during movements.

The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push med wrist brace splint can be washed normally.

INDICATIONS

- Strain complaints in the wrist extensors
- Rheumatoid complaints
- Carpal tunnel syndrome
- RSI

There are no known disorders with which the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly at first.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING

The wrist brace is available in 4 sizes. Measure the circumference of the wrist, as shown in the illustration on the packaging. The table on the packaging shows the correct brace size for your wrist size.

- A. The correct fitting sequence is determined by the increasing molecule symbols on the hook and loop fasteners.
- Slide the base around the hand until it reaches the thumb.
- Use the non-elastic strap to set the required compression around the wrist.
- B. Close the brace around the lower arm and hand.

ADJUSTING THE SPOON

If the spoon needs to be adjusted, this should always be done by a specialist. You are strongly advised against adapting the spoon yourself.

The spoon is made from ThermoLyn®, a synthetic material that can be shaped both in a cold and in a warm state. This makes it easy to achieve a functionally optimal position of the wrist for each individual. The easiest way to shape the spoon is by using a hammer.

To shape the spoon in a warm state, the material must be heated to approximately 160 °C, either in an oven or with a hair-drier.

WASHING INSTRUCTIONS

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30 °C, on a delicate fabric cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

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