## PRODUCT INFORMATION

**PUSH MED WRIST BRACE SPLINT**

**INTRODUCTION**

Push med products provide effective solutions in the case of a disorder or reduced functioning of the locomotor apparatus.

Your new wrist brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push Splint improves stability and provides pain relief. This increases physical strength. Mechanical support, adjustable pressure and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote safe use.

Read the product information carefully before you use the Push med wrist brace splint. If you have any questions, please consult your reseller or send an e-mail to [info@push.eu](mailto:info@push.eu).

**PRODUCT CHARACTERISTICS**

Your Push med wrist brace splint offers a high level of support, protection and relaxation of the wrist, without restricting the functionality of the hand. The brace is easily fitted with one hand around the lower arm, the wrist and the hand. A circular, non-elastic connection ensures that the adjustable splint tightly fits the anatomy of the wrist. This restricts palmary flexion and places the hand in a functional position.

So-called comfort zones of padded material have been created around vulnerable areas of the wrist. A proper anatomical fit contributes to the brace remaining in position during movements.

The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside felt soft and keeps the skin dry. The Push med wrist brace splint can be washed normally.

**INDICATIONS**

- Strain complaints in the wrist extensors
- Rheumatoid complaints
- Carpal tunnel syndrome
- RSI

There are no known disorders where the brace cannot or should not be worn.

In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

**DETERMINING SIZE / FITTING**

The wrist brace is available in 4 sizes. Measure the circumference of the wrist, as shown in the illustration on the packaging. The table on the packaging shows the correct brace size for your wrist size.

A. The correct fitting sequence is determined by the increasing molecule symbols on the hook and loop fasteners.

Slide the base around the hand until it reaches the thumb.

Use the non-elastic strap to set the required compression around the wrist.

B. Close the brace around the lower arm and hand.

**ADJUSTING THE SPOON**

If the spoon needs to be adjusted, this should always be done by a specialist. You are strongly advised against adapting the spoon yourself.

The spoon is made from ThermoLyn®, a synthetic material that can be shaped both in a cold and in a warm state. This makes it easy to achieve a functionally optimal position of the wrist for each individual. Using a hammer to tap the spoon into the desired shape, is the easiest solution.

To shape the spoon in a warm state, the material must be heated to approximately 160°C, either in an oven or with a hair-drier.

**WASHING INSTRUCTIONS**

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, observing the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and air the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30°C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

**IMPORTANT**

- Proper use of the Push brace may require diagnosis by a doctor or therapist. Please be aware that an ill-fitting brace does not function properly.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before purchasing, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to be able to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace each time before use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- Within the framework of EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the competent authority of the Member State in which the user is established.

Push® is a registered trademark of Nea International bv

For details of the distributor in your country, we refer to [www.push.eu/distributors](http://www.push.eu/distributors)

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