PUSH MED ANKLE BRACE AEQUI FLEX

INTRODUCTION

Push med products provide effective solutions for the treatment and prevention of injuries of the locomotor apparatus.

Your new ankle brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med Aequi flex increases ankle stability and helps control pain. This increases physical strength. Mechanical support, adjustable pressure and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push Aequi flex. If you have any questions, please consult your reseller or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

The Push Aequi flex is an effective ankle brace, which strikes a balance between efficient mechanical support of an unstable ankle and good wearar comfort. The rigid support is situated exclusively on the medial (inner) side of the ankle and has a cavity for the malleolus. The lateral side is made from comfortable and elastic foam that easily adapts to the individual anatomy. The bracing material is constructed in such a way that it is rigid in the centre, becoming softer towards the edges to ensure a good fit and wearar comfort. The brace is fastened by means of a diagonal strap. The elastic straps apply pressure around the ankle. The Push Aequi flex can be used in the functional treatment of ankle injuries and as a long term support in cases of chronic ankle instability. The inversion/eversion movement (inward / outward movement of the foot) is restricted while plantar/dorsal flexion (bending and stretching) is allowed, for adequate freedom of movement. This means that no inhibition is felt during normal walking and running. The numbers on the straps ensure easy fitting. The Push Aequi flex is worn over the sock, and optimal stabilisation can only be achieved if it is worn with a closed (lace-up) shoe.

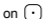
INDICATIONS

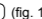
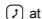
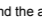
- Acute lateral ankle ligament trauma
- Follow-up treatment of conservative or operated ankle fractures
- Moderate to serious instability
- Moderate to serious arthrosis
- Secondary prevention of ankle ligament trauma

There are no known disorders with which the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

FITTING

The hook and loop fasteners are marked with symbols. These are to help you fasten the hook and loop material in the correct sequence. First , then , then  etc.

- The Push Aequi flex is worn over the sock
- Place the brace around the foot. The rigid part is on the inside of the foot. Put the long straps on the sides
- Fasten the diagonal strap marked with  (fig. 1)
- Take hold of the elastic strap marked with  at the outside of the foot. Pull it diagonally across the instep, wrap it around the ankle just above the heel and fasten in place (fig. 2)
- Finally take hold of the elastic strap on the inside of the foot, marked with . Pull it from the instep across the dorsum (top of the foot), wrap it around the ankle over the first strap and fasten in place (fig. 3).

WASHING INSTRUCTIONS

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

The brace can be washed by hand at temperatures up to 30°C. Do not use any bleaching agents.

Hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Try on the brace before purchasing to check its size. Your retailer's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace before each use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- To promote the useful life of the brace, it should not be exposed to temperatures above 70°C.
- In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the appropriate regulatory body in the user's country of residence.
- Maximum stabilisation is achieved when the brace is worn inside a closed (lace-up) shoe. Be careful not to slip when the brace is worn without a shoe.

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