PUSH MED KNEE BRACE

INTRODUCTION

Push med products provide solutions in the case of a disorder or reduced functioning of the locomotor apparatus.

Your new knee brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med knee brace increases knee stability and helps control pain. This increases physical strength. Mechanical support, adjustable compression and multiple features that keep the brace in place are the foundation of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and greater freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push med knee brace. If you have any questions, please consult your retailer or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

Your Push med knee brace offers a high level of excellent support and protection to the knee joint.

The knee joint is supported in the medial-lateral direction by innovative leaf spring joints hinges and static straps. This allows for unobstructed flexion and extension of the knee. The hinges follow the natural bending point of the knee.

The silicone application stimulates the proprioception of the knee joint, to adequately anticipate any imminent trauma.

So-called "comfort zones" of padded material have been created around vulnerable areas of the knee. A proper anatomic fit and the silicone inlays contribute to keep the brace in place during movement. Its small volume enables the brace to be worn underneath clothing without being noticed.

The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push med knee brace can be washed normally.

INDICATIONS

- Arthrosis with instability
- Rheumatoid complaints
- Collateral ligament trauma
- Residual instability after collateral ligament trauma
- Postoperative follow-up treatment

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly at first.

Pain is always a warning sign. If the pain persists or symptoms worsen, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING

The knee brace is available in 5 sizes. Sit with your knee bent at 90°. Then measure the circumference around the narrowest part of your leg below the knee joint. The chart on the packaging shows the correct brace size for your knee.

- Slide the brace over the knee joint. To do this, hold the brace by the plastic tabs above the hinges and pull it up the leg. Ensure that all the straps are loose. Fit the circular buttress properly around the patella (knee cap).
- Close the straps at the back and the front of the thigh. Varying the length at the back and the front allows you to optimally position the stiffeners on the side of the leg, so that the brace fits snugly and works perfectly.
- Then close the straps that crisscross at the front of the shin.

WASHING INSTRUCTIONS

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30 °C, on a delicate fabric cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used; after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist. Please be aware that an ill-fitting brace does not function properly.
- Push braces were designed to be worn on undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Try on the brace before purchasing, to check its size. Your retailer's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace before each use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the appropriate regulatory body in the user's country of residence.

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