LIGADURA DE RÓTULA PUSH MED

INTRODUÇÃO

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PUSH MED PATELLA BRACE

INTRODUCTION

Push med products provide effective solutions in the case of a disorder or reduced functioning of the locomotor apparatus.

Your new patella brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med patella brace improves functioning and provides pain relief. This increases physical strength. Mechanical support, adjustable pressure and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Zur the product information carefully before you use the Push med patella brace. If you have any questions, please consult your reseller or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

The Push med patella brace applies local pressure on the knee tendon and supports the patella. This relieves the insertion of the thigh muscle and reduces the tractive force on the lower leg.

The pad consists of a skin-compatible and elastic polymer. The synthetic bracing element underneath the pad ensures an even distribution of pressure.

A proper anatomic fit and the material of the pad contribute to the brace remaining in place during walking. The brace is made of Sympress™, a high-quality comfortable material. The use of microfibras makes the inside feel soft and keeps the skin dry.

The Push med patella brace is easy to apply and can be washed without any problem.

INDICATIONS

- Tendinitis of the patella tendon
- Osgood-Schlatter disease
- Patellofemoral pain syndrome
- Subluxation/hypermobility proximal tibiofubular joint

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING

The patella brace is available in 1 size. The measurements range from 28 to 45 centimetres.

Ensure that the pressure point of the brace (at the level of the logo) is located immediately underneath the kneecap.

Tension the adjustable strap and close it.

WASHING INSTRUCTIONS

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, observing the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30 °C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before purchasing, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to be able to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace each time before use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.

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Push® is a registered trademark of Nea International bv