PUSH MED BACK BRACE

INTRODUCTION

Push Med products provide effective solutions in the case of a disorder or reduced functioning of the locomotor apparatus.

Your new back brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med back brace offers pain relief by supporting the lumbar vertebral column. This increases physical strength. Mechanical support, adjustable compression and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push med back brace. If you have any questions, please consult your retailer or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

Your Push med back brace ensures a proper position of the vertebral column by means of metal bracing elements adjusted to the anatomy of your back. The brace supports the back from the sacral vertebrae to the lower thoracic vertebrae (S2 to T8). Tensioning the elastic straps provides compression and increases the support offered by the stiffeners.

Because of its design and the materials used, the brace is comfortable to wear during standing or sitting activities, both at home and at work. Its anatomical fit is suitable for both men and women and provides appropriate position retention. The close body-fit of the brace renders it invisible underneath your clothes. Compression can be adjusted easily to suit your preferences. The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the back part feel soft and keeps the skin dry. Microperforations allow the outer material to breathe. The Push med back brace is easy to apply and can be washed without any problem.

INSTRUCTIONS FOR INDIVIDUAL FITTING BY A PROFESSIONAL SPECIALIST

The specialist determines whether the shape of the stiffeners needs to be adjusted for you. The shape of the metal stiffeners can be adjusted to the individual user by bending with force.

INDICATIONS

- Spondylolysis
- Spondylolisthesis
- Lumbale Arthrose
- Lumbar arthrosis
- Discopathy
- Osteoporotic lumbar impression fractures
- Postsurgical treatment

There are no known disorders with which the brace cannot or should not be worn. Avoid applying the brace too tightly at first.


Pain is always a warning sign. If pain persists or symptoms worsen, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING

The back brace is available in 6 sizes. The size is determined by the circumference of the waist. The table on the packaging shows the correct brace size for your waist.

- A. Wear the back brace against the skin or over your undergarments. Please ensure that the brace is positioned in the centre of the vertebral column at the level of the lower back. The bottom of the brace covers the hips. The shape of the bracing elements must fit your back when standing up with a straight back.
- B. Loosen the elastic straps before fitting the brace. Hold the brace by the lower edge and close the hook and loop fastener or use the grips.
- C. Tension and close the elastic straps. If the pressure experienced is too tight, the brace should be slackened. If the brace has been fitted properly, you should feel a comfortable, straight position.

WASHING INSTRUCTIONS

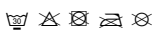To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and hang the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry. Before washing, open the zip and remove the metal bracing elements from the brace.

The brace can be washed at temperatures up to 30 °C, on a delicate fabric cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used; after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before purchasing, to check its size. Your retailer's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace before each use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the appropriate regulatory body in the user's country of residence.

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