PUSH MED SHOULDER BRACE PLUS

INTRODUCTION

Push med products provide effective solutions for the treatment and prevention of injuries of the locomotor apparatus.

Your new shoulder brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med shoulder brace Plus ensures proper fixation and pain relief. Mechanical support and stable shoulder positioning constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. We are committed to providing you maximum confidence in the interest of user satisfaction. Read the product information carefully before you use the Push med shoulder brace Plus. If you have any questions, do not hesitate to ask your supplier or use www.push.eu (info@push.eu).

PRODUCT CHARACTERISTICS

Your Push med shoulder brace Plus consists of four components. The breast strap, which is closed around the chest or upper waist, constitutes the basis of the brace. This strap only needs to be adjusted once for the correct size. The "quick lock" can be used to open and close the breast strap and to adjust it for maximum comfort.

The upper arm strap fixes the upper arm against the body to prevent abduction of the shoulder as much as possible. The comfortable wrist-hand support keeps the arm in place and supports it. The added shoulder strap adds comfort to the Push med shoulder brace Plus: this strap provides additional immobilization and support to the affected shoulder joint and allows the hand to be kept at the desired height. Each fixation point is individually adjustable, for an optimal fit in all circumstances. The brace is made of Sympress™, a comfortable and high-quality material. The microfiber used makes the inside soft to the touch and keeps the skin dry. The edges of the straps also feel soft. The Push med shoulder brace Plus can be washed normally.

INDICATIONS

- Collarbone fracture
- Treatment after (sub)luxation
- Treatment after (sub)luxation of the SC or AC joint
- Postoperative treatment
- Humeroacropular periarthritis
- Subacromial impingement syndrome
- As a sling in the case of a facciali hemiparesis (also at night)

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too lightly at first.

Pain is always a warning sign. If the pain persists or symptoms worsen, it is advisable to consult your doctor or therapist.

FITTING / SETTING

The shoulder brace is available in 3 sizes. For proper fitting, the chest circumference is measured 5 centimetres above above the elbow, with your arm hanging relaxed by your side. The table on the packaging shows the correct brace size for your chest circumference.

Use the molecule symbols featured on the hook and loop fasteners as your guide to determine the correct fitting sequence (☐ ☐ ☐ ☐).

All hook and loop fasteners marked with symbols must be opened, the others remain closed.

- A. Grasp the brace with the unaffected arm at the wide section of the shoulder strap.
- B. Put the affected arm and your head through the opening.
- C. Close the upper arm strap by passing the strap through the clasp and closing it just above the elbow (☐).

For the professional who supplies the brace. Before initial use, two straps of the brace must be adjusted for the patient once.

D1. Adjust the shoulder strap at the back to ensure that the breast strap is at the chest level as the upper arm.

D2. Then adjust the breast strap so that it fits closely to the body and feels comfortable for the patient.

- D. Close the breast strap so that it fits comfortably (☐).
- E. Put the hand in the wrist-hand support and close it on the shoulder strap (☐).
- F. You may slide the wrist-hand support on the breast strap (left/right) or pull it (up/down), to achieve the best possible fit.

- Fitting instructions for patients in a recining position:
- Measure half of the circumference of the patient
 - Multiply this by 2
 - Use the clasp to adjust the length of the breast strap at the back
 - Open the breast strap using the quick release
 - Fit the brace

CARE AND MAINTENANCE

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage. Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30°C, on a delicate fabric cycle or by hand. Do not use any bleaching agents. A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Try on the brace before purchasing, to check its size. Your retailer's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace before each use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the appropriate regulatory body in the user's country of residence.
- It is advisable in the case of possible skin irritation to wear the brace over one or other's clothing.

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