PUSH MED SHOULDER BRACE

INTRODUCTION

Push med products provide effective solutions for the treatment and prevention of injuries of the locomotor apparatus.

Your new shoulder brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med shoulder brace offers proper fixation and pain relief. Mechanical support and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. The aim is to provide optimal security. In the interest of satisfied users. Read the product information carefully before you use the Push med shoulder brace. If you have any questions, do not hesitate to ask your supplier or use www.push.eu (info@push.eu).

PRODUCT CHARACTERISTICS

Your Push med shoulder brace consists of three components. The breast tulle, which is closed around the lower chest or upper waist, constitutes the basis of the brace. This strap only needs to be adjusted once for the correct size. The upper arm strap ensures fixation of the upper arm against the body, preventing use of the shoulder. The position of the shoulder brace can be set in such a way as to fix the arm in a particular position. The position can be changed easily by turning the breast strap slightly. The third fixation point is the comfortable wrist fixation and hand support. This keeps the arm fixed and prevents the hand from sliding down while giving it support. If only abduction restriction is required, both the wrist fixation and the hand support may be removed. As the brace closes around the body, the neck and shoulder regions remain unstressed. Each fixation point is individually adjustable, for an optimal fit in all circumstances.

The Push med shoulder brace can be washed normally.

INDICATIONS

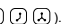
- Collarbone fracture
- Treatment after (sub)luxation
- Treatment after (sub)luxation of the SC or AC joint
- Subscapular humerus fracture
- Humeroscapular periarthritis
- Postoperative treatment
- As a sling in the case of a flaccid hemiparesis (also at night)

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

FITTING / SETTING

The shoulder brace is available in 3 sizes. For each size, the model can be used for both the left and the right shoulder. For proper fitting, the chest circumference is measured 5 centimetres above the level of the hanging elbow. The table on the packaging shows the correct brace size for your chest circumference.

The correct fitting sequence is determined by the increasing molecule symbols on the hook and loop fasteners (  ).

Before the brace is fitted, it is recommended to attach the various braces. In order to get a good overview of the brace.

A. Close the breast strap so that it fits comfortably ().

For the professional who supplies the brace. Before initial use, the strap of the brace must be adjusted for the patient once.

A1. Then adjust the breast strap so that it fits closely to the body and feels comfortable for the patient.

B. Close the upper arm strap by passing the strap through the clasp and closing it just above the elbow ().

C. Close the wrist fixation (). You may slide the wrist fixation and hand support on the breast strap to the left or to the right, to achieve the best possible position.

Fixing instructions for patients in a reclining position:

- Measure half of the circumference of the patient
- Multiply this by 2
- Use the clasp to adjust the length of the breast strap at the back
- Open the breast strap using the quick release
- Fit the brace

CARE AND MAINTENANCE

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, observing the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash after try the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30° C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before purchasing, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to be able to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace each time before use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be reached if the product is fully intact.
- Within the framework of EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the competent authority of the Member State in which the user is established.
- It is advisable in the case of possible skin irritation to wear the brace over one's clothes.

Push® is a registered trademark of Nea International bv

For details of the distributor in your country, we refer to www.push.eu/distributors

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