## PUSH MED NECK BRACE

### INTRODUCTION

Push med products provide effective solutions in the case of a disorder or reduced functioning of the locomotor apparatus.

Your new neck brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med neck brace provides pain relief by stabilising the cervical vertebrae. Mechanical support and good anatomic fit constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and enhanced functionality. To promote satisfied use.

Read the product information carefully before you use the Push med neck brace. If you have any questions, please consult your reseller or send an e-mail to [info@push.eu](mailto:info@push.eu).

### PRODUCT CHARACTERISTICS

Your Push med neck brace provides support to the vertebral column by its anatomic shape and the use of semi-rigid foam. The neck brace contains a detachable element for additional support. This element rests on the breastbone and the jaws, for highly effective support. The brace can also be used without the reinforcement element if lighter support is desired. This may be the case when sleeping or as the treatment progresses. A recess is left for the Adam's apple to enhance comfort.

The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push med neck brace is easy to apply and can be washed without any problem.

### INDICATIONS

- Follow-up treatment of whiplash
- Acute neck injuries
- Postoperative Nachbehandlung
- Postsurgical treatment
- Cervical arthrosis
- Cervical arthrosis with a pseudoradicular complaints
- Tension headache

There are no known disorders where the brace cannot or should not be worn. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

### DETERMINING SIZE / FITTING

The neck brace is available in 2 circumferences and 2 heights. Hold the head straight and measure the circumference of the neck from the back over the Adam's apple. The table on the packaging shows the correct brace size for your neck circumference. The required height is determined during fitting. The head is properly supported when it is in a neutral position.

A. Hold the brace in both hands in front of you, so that the inside is visible. The point at the centre of the brace should be pointing down.

B. Then close the hook and loop fastener at the back of the neck.

Removing and fitting the reinforcement element:

Place the brace with the inside on a solid surface, with the dot in the centre of the brace pointing towards you. There is a zip on the right-hand side.

Open the zip to remove the reinforcement element. Locate the aperture in the reinforcement element and pull the element out of the brace, while holding the brace tightly. To insert the reinforcement element, perform the actions in reverse order.

### WASHING INSTRUCTIONS

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, observing the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry. Remove the reinforcement element from the brace before washing.

The brace can be washed at temperatures up to 30°C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

### IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist. Please be aware that an ill-fitting brace does not function properly.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before purchasing, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to be able to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace each time before use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- Within the framework of EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the competent authority of the Member State in which the user is established.

Push® is a registered trademark of Nea International bv

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