PUSH MED NECK BRACE

INTRODUCTION

Push med products provide effective solutions in the case of a disorder or reduced functioning of the locomotor apparatus.

Your new neck brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med neck brace provides pain relief by stabilising the cervical vertebrae. Mechanical support and good anatomic fit constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and enhanced functionality. To promote satisfied use.

Read the product information carefully before you use the Push med neck brace. If you have any questions, please consult your retailer or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

Uw Push med neck brace provides support to the cervical spine by its anatomic shape and the use of semi-rigid foam. The neck brace contains a detachable element for additional support. The plate provides support in three places between the jaw and the sternum, ensuring effective stability of the cervical spine. The brace can also be used without the reinforcement element if lighter support is desired. This may be the case when sleeping or as the treatment progresses. A recess is left for the Adam's apple wire eine Aussparung gelassen, um den Komfort zu erhöhen.

The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push med neck brace is easy to apply and can be washed without any problem.

INDICATIONS

- Follow-up treatment of whiplash
- Acute neck injuries
- Postoperative Neckbehandlung
- Cervical arthrosis
- Cervical arthrosis with a pseudoradicular complaints
- Tension headache

There are no known disorders where the brace cannot or should not be worn. Avoid applying the brace too tightly at first.

Pain is always a warning sign. If the pain persists or symptoms worsen, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING

The neck brace is available in 2 circumferences and 2 heights. Hold the head straight and measure the circumference of the neck from the back over the Adam's apple. The table on the packaging shows the correct brace size for your neck circumference. The required height will be determined during fitting. The head is properly supported when it is in a neutral position.

A. Hold the brace with both hands in front of you, so that the inside is visible. The point at the centre of the brace should be pointing down.

B. Then close the hook and loop fastener at the back of the neck.

Removing and fitting the reinforcement element:

Place the brace with the inside on a solid surface, with the dot in the centre of the brace pointing towards you. There is a zip on the right-hand side.

Open the zip to remove the reinforcement element. Locate the aperture in the reinforcement element and remove the element from the brace while holding it tightly. To insert the reinforcement element, perform the actions in reverse order.

WASHING INSTRUCTIONS

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry. Remove the reinforcement element from the brace before washing.

The brace can be washed at temperatures up to 30 °C, on a delicate fabric cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Try on the brace before purchasing, to check its size. Your retailer's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace before each use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the appropriate regulatory body in the user's country of residence.

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