PUSH MED ELBOW BRACE EPI

INTRODUCTION

Push med products provide effective solutions for the treatment and prevention of injuries of the locomotor apparatus.

Your new elbow brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med elbow brace Epi improves functioning and provides pain relief. This increases physical strength. Mechanical support, adjustable pressure and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push med elbow brace Epi. If you have any questions, please consult your retailer or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

Your Push med elbow brace Epi offers local pressure on the extensors (tennis elbow) or the flexors (golfer's elbow) of the wrist. This relieves the strain where these muscles come in contact with the elbow joint. The brace can easily be fitted and closed around the lower arm with one hand. The bracing element around the arm ensures proper distribution of pressure. The padding consists of a skin-compatible and elastic polymer. A proper anatomic fit and the material of the padding ensures that the brace remains in place during movement.

The brace is made of Sympress™, a high-quality and comfortable material. The use of microfibrres makes the inside feel soft and keeps the skin dry. The Push med elbow brace Epi can be washed normally.

INDICATIONS

- Tennis elbow (lateral epicondylitis)
- Golfer's elbow (medial epicondylitis)

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly at first.

Pain is always a warning sign. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING

The Elbow brace Epi is available in 1 size. The measurements range from 20 to 35 centimetres.

Slide the brace onto the arm. For tennis elbow, the pad should be positioned on the outside of the arm; for golfer's elbow, it should be on the inside. Do not position the strap ring in the crease of the elbow.

Place the brace on the forearm muscles, around six centimetres from the condyle (the outer protrusion on the elbow).

Tighten the strap so that you feel pressure when your arm is bent and your muscles are tensed. You should not feel any tension when your arm is relaxed. The strap can be left in the strap ring when the brace is removed.

WASHING INSTRUCTIONS

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30°C, on a delicate fabric cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Try on the brace before purchasing, to check its size. Your retailer's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to be able to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace before each use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the appropriate regulatory body in the user's country of residence.

Push® is a registered trademark of Nea International bv

For details of the distributor in your country, refer to www.push.eu/distributors

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