PUSH ORTHO ANKLE BRACE AEQUI

INTRODUCTION

Push ortho products provide effective solutions in the case of a disorder or reduced functioning of the locomotor apparatus.

Your new ankle brace is a Push ortho product, characterised by a high level of effectiveness, comfort and quality. The Push Aequi increases ankle stability and helps to control pain. This increases physical strength. Mechanical support, adjustable pressure and good position retention constitute the basis of its performance.

Push ortho products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push Aequi. If you have any questions, please consult your reseller or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

The Push Aequi is an effective ankle brace, which strikes a balance between efficient mechanical support of an unstable ankle and good wearer comfort. The rigid support is situated exclusively on the medial (inner) side of the ankle and has a cavity for the malleolus. The lateral side is made from comfortable foam, which easily adapts to the individual anatomy. The bracing material is constructed in such a way that it is very rigid in the centre, becoming softer towards the edges to ensure a good fit and wearer comfort. The brace is fastened by means of a diagonal strap. The elastic straps apply pressure around the ankle. The Push Aequi can be used in the functional treatment of ankle injuries and as a long term support in cases of chronic ankle instability. The inversion/eversion movement (inward / outward movement of the foot) is restricted while plantar/dorsal flexion (bending and stretching) is allowed, for adequate freedom. This means that no inhibition is felt during normal walking and running. The numbers on the straps make fitting the brace simply itself. The Push Aequi is worn over the sock, and optimal stabilisation can only be achieved if it is worn with a closed (lace-up) shoe.

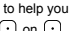
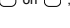
INDICATIONS

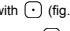
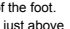
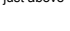
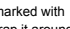
- Moderate to serious instability
- Lateral ankle ligament injury
- Follow-up treatment of conservative or operated ankle fractures
- Moderate to serious arthrosis
- Secondary prevention of ankle ligament injury

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too lightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

FITTING

The hook and loop fasteners are marked with symbols. These are to help you fasten the hook and loop fasteners in the correct sequence. First , then  on  etc.

- The Push Aequi is worn over the sock.
- Place the brace around the foot. The rigid part is on the inside of the foot. Put the long straps to the sides.
- Fasten the diagonal strap marked with  (fig. 1).
- Take hold of the elastic strap marked with  at the outside of the foot. Pull this diagonally across the instep, wrap it around the ankle just above the heel and fasten in place (fig. 2).
- Nehmen Sie schließlich das elastische Band auf der Innenseite des Fußes, das Band mit . Ziehen Sie dieses quer über den Spann, wickeln Sie es fest (Abb. 2).
- Finally take hold of the elastic strap on the inside of the foot, marked with . Pull this from the instep across the dorsum (top) of the foot, wrap it around the ankle over the first strap and fasten in place (fig. 3). The instep is the lower part of the foot, where the strap starts.

WASHING INSTRUCTIONS

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, observing the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

The brace can be washed by hand at temperatures up to 30° C. Do not use any bleaching agents.

Hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

IMPORTANT

- The efficient use of the Push Aequi depends on the correct diagnosis of the causes of pain or instability. Always check first with your doctor, a physiotherapist or another specialist before using a brace.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before purchasing, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to be able to consult it for fitting instructions whenever needed.
- Check your Push brace each time before use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- To promote the useful life of the brace, it should not be exposed to temperatures above 70° C.
- Within the framework of EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the competent authority of the Member State in which the user is established.
- Maximum stabilisation is achieved when the brace is worn inside a closed (lace-up) shoe. Be careful not to slip when the brace is worn without a shoe.

Medical Device

11/2021

I 3039-Z 0 / 94

Made in the EU

Designed in the Netherlands

www.push.eu

European 6199 AB Maastricht-Airport

Nea International bv

3.201

ORTHO

ANKLE BRACE AEQUI

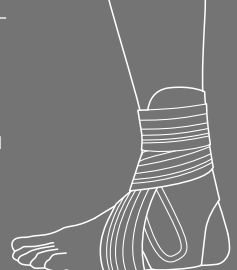
KNÖCHELORTHOSE AEQUI

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ENKELBRACE AEQUI

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TOBILLERA AEQUI

PRODUCT INFORMATION

EN • DE • NL • FR • IT

ES • PT • CS • PL • HU

JA • SV • DA • NO • FI

For details of the distributor for your country, we refer to www.push.eu/distributors.

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