PUSH ORTHO HAND ORTHOSIS DPT

INTRODUCTION

Push ortho products offer solutions for conditions involving reduced function of the musculoskeletal system.

Your new hand orthosis is a Push ortho product characterised by a high degree of effectiveness, comfort and quality. The Push ortho Hand Orthosis DPT supports the treatment of your condition.

Push ortho products are innovative and carefully made for you. With a view to ensure maximum ease of use and freedom of movement.

Before using this product, please read the product information carefully and follow the instructions below. If you have any questions, please contact your supplier or send an email to info@push.eu.

PRODUCT CHARACTERISTICS

Dupuytren's disease is a progressive condition. This orthosis is intended to help reduce the corresponding symptoms, but cannot cure the condition itself. After stopping therapy with the orthosis, the progression of the condition may return.

The Push ortho Hand Orthosis DPT is used to treat a finger that is bent by connective tissue strands, a so-called contracture, caused by Dupuytren's disease. The product is not suitable for the thumb. The goal is to reduce the degree of contracture or prevent further development of contracture. The orthosis boasts a specially developed leaf spring system incorporated into a comfortable glove; it consists of two components: a base part and a finger part. The orthosis exerts a continuous, targeted and slight stretching force on the connective tissue strands. This force can be adjusted so the orthosis can be worn for longer periods. The orthosis can be applied to one or more affected fingers per hand (with the exception of the thumb). While wearing the orthosis it is still possible to bend the fingers occasionally and perform light actions. Moving the fingers also prevents stiffness and promotes blood circulation. You will be advised to use this orthosis at prescribed times as a part of your therapy.

The healthcare professional who provides you with the product can, if necessary, adjust the orthosis to tailor the extension force to meet your individual needs. It is therefore very important that you follow the advice of your healthcare professional and have follow-up consultations where necessary. If you have any problems, or there are changes in your situation, it will be necessary to make an appointment with your healthcare professional.

Hand hygiene is very important. You can use a washing machine to wash the textile parts of the orthosis at 30 °C.

INDICATIONS

Morbus Dupuytren in the fingers (not the thumb) in the following stages:

- Tubiana stage 1 and 2
- Tubiana stage N for active strands

The orthosis can be used in the following situations after assessment by a doctor:

- To reduce a contracture as primary therapy
- To reduce a (residual) contracture to support another treatment
- To slow the progression of an incipient or existing contracture
- To maintain the results of treatment of a contracture

CONTRAINDICATIONS

Do not use the orthosis, or discontinue use of the orthosis, in the following situations:

- Pain such that the orthosis cannot be worn for more than 1.5 hours, even on the lightest setting
- Peripheral neuropathy, resulting in insufficient sensitivity to pain perception
- Absence of the distal phalanx
- Non-blanchable redness or redness that takes more than 24 hours to disappear at the lightest setting.
- Immediately after an operation. The orthosis must only be used when the wound can be mechanically loaded, and then at the lightest setting to treat any residual contracture.

FITTING

The base around the middle hand is for the attachment of one or more finger parts. The finger part has a leaf spring system in a comfortable cover. The leaf spring system has a soft support block that is placed on the back of the finger. The finger part is attached to the back of the hand with a hook-and-loop fastener.

The hook-and-loop fasteners are marked with symbols. These help you close the fasteners in the right order.

First  on , then  on .

1. Open the straps of the base part and place your hand in the base part (drawing 1).
2. Close the base part comfortably around the wrist with the hook-and-loop closure  (drawing 2).
3. Make sure the edge of the base part aligns with the finger knuckles (drawing 3).
4. If you received multiple finger parts, check which finger part is intended for which finger using the marking on the finger part. Slide your finger into the finger part and then bend the part so you can place the block on the middle of your first phalanx (counting from the hand) (drawing 4).
5. The textile on the inside of the finger should reach the fold between the first and second phalanx (drawing 5).
6. Position the finger part with the hook-and-loop on the base, in line with the finger. If necessary, repeat this for each finger you have received a finger part for (drawing 6a).

Should the support block in the middle of the finger part tilt away from the finger, it may be necessary to correct the attachment of the finger parts. A correction strap is provided for this (see drawing 6b). It can be attached at a certain angle so the finger part remains stable and does not tilt while bending. Make sure the bottom strap remains attached to the finger part. This setting can be explained by the specialist when you get the orthosis.

7. Close the elastic strap  around the wrist to fix the bottom of the finger part or parts (drawing 7).
8. Now close the elastic strap  over the back of the hand and thus fix the finger part or parts (drawing 8a and 8b).

MAINTENANCE INSTRUCTIONS

To promote hygiene and maximize the lifespan of the orthosis, it is important to wash it regularly by following the washing instructions.

If you are not going to use the orthosis for a long period of time, it is advisable to wash and dry it before storage.

Please note: the leaf spring system in the finger parts must be removed before washing the textile part (drawing A).

You can remove the leaf spring system from the finger part as follows:

Loosen the two elastic straps attached with the hook-and-loop fastener and lay them open (drawing B)

Grasp the support block, bend the spring package on the hook-and-loop side, and pull the spring package out on that side (drawing C).


Note: the spring package can snap out of the textile part. This isn't a problem, but you can avoid it by carefully pulling the spring pack out.

The leaf spring system can then be pulled out from the other side (drawing D).

When replacing the leaf spring system, the sticker with the finger shows that this part must be inserted on the finger side. Then bend the spring package on the other side and slide it back in.

If you have received multiple finger parts, it is important to pay attention to which leaf spring system fits which textile part and which finger part is intended for which finger. This can be checked by using the marking written by the specialist on the textile part, as well as on the sticker of the leaf spring system. (See drawing D).

WASHING INSTRUCTIONS

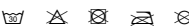Remove the leaf spring system from each finger section before washing. Before washing, close the hook-and-loop straps to protect the orthosis and other laundry. The orthosis can be washed at up to 30 °C on a delicate wash, or it can be washed by hand. Do not use any bleaching agents.

The orthosis may be open in the washing machine on a light setting; it should then be hung up to dry in the open air (not near a heater, and never in a tumble dryer).

IMPORTANT INFORMATION

- Correct diagnosis and a good treatment plan are necessary for correct use of the Push ortho Hand Orthosis DPT. Always consult a specialist before using the orthosis.
- Carefully follow the wearing times prescribed by your medical professional or specialist.
- To stimulate blood circulation and prevent stiffness, it is advisable to occasionally bend and straighten the finger(s) while wearing. To ensure the orthosis works optimally, keep the finger in a relaxed, stretched position as much as possible.
- The medical professional or specialist will check whether any adjustments to the product are required during a consultation.
- If the finger part shifts or rotates during use, the orthosis is not optimally positioned around your hand and finger. Should you be affected by this, please re-read the fitting instructions thoroughly. If necessary, consult the specialist who supplied the orthosis to you.
- The correct choice of size is essential for optimal functioning. This will be determined by the specialist.

- Always keep the product information so you can read over the fitting instructions again later.
- The extra parts supplied with the orthosis – extra leaf springs – must be stored carefully. They may be required in the further treatment process.
- Check for signs of wear or ageing on any parts or seams on your orthosis before every use. Optimal function of your orthosis is only guaranteed if the product is completely intact.
- In accordance with EU regulation 2017/745 concerning medical devices, the user must report any serious incident related to the product to the manufacturer and the competent authority in the EU Member State in which the user is based.
 - If after a few days it is not possible to wear the orthosis for at least 1.5 hours due to pain, you must discontinue use. The orthosis setting may be too heavy, and it will require adjustment by the specialist.

- Stay alert for redness at the location of the support block:
 - Do not wear the orthosis if you see redness.
 - Redness that turns white when gentle pressure is applied and disappears within an hour is normal.
 - Redness that disappears under applied pressure lasting for more than an hour (but less than 24 hours) is not an immediate cause for concern. If redness is a concern after getting used to the product, it is wise to contact your medical professional or specialist.
 - If the red skin colour does not turn white when gentle pressure is applied, or the redness takes more than 24 hours to disappear, contact your medical professional or specialist. A less demanding setting or adjustment of the wearing times may be a solution. If this does not help, this orthosis is probably unsuitable for you.

- Consult your healthcare professional if the location and area of redness does not match that of the pad, or if it is accompanied by itching or flaking.
- Contact your medical professional or specialist if you experience persistent tingling or numbness. A less demanding setting or adjustment of the wearing times may be a solution.
- If the fingerloop overextends when wearing the orthosis, your medical professional or specialist may advise you to support it with a tape or other aid.
- It is possible to wear the orthosis (for a part of the night if you meet the following conditions:
 - you are completely used to the orthosis;
 - you can easily take the orthosis off without thought;
 - you have no complaints concerning the nerves or blood vessels in your fingers.

Always discuss this with your medical professional or specialist first.

The Push Hand Orthosis DPT was designed to be worn on undamaged skin. If the skin is damaged, it must first be covered with appropriate material.

- Changes to the settings of the leaf spring system may only be made by a medical professional or specialist. Instructions are available to the medical professional or specialist via a QR code on a set of leaf springs included in the packaging of the finger part.

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