PUSH CARE NECK BRACE

INTRODUCTION

Push care products provide effective solutions when your freedom of movement is restricted by an injury or disorder. Approved by physicians and scientists.

Your new neck brace is a Push care product, characterised by a high level of effectiveness, comfort and quality. The Push care neck brace provides pain relief by stabilising the cervical vertebrae. Mechanical support and good anatomic fit constitute the basis of its performance.

Push care products are innovative and are manufactured for you with great care. This ensures maximum security and carefree use.

Read the product information carefully before you use the Push care neck brace.

If you have any questions, please consult your reseller or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

Your Push care neck brace provides support to the vertebral column by its anatomic shape and the use of semi-rigid foam. The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push care neck brace is easy to apply and can be washed without any problem.

INDICATIONS

- Complaints caused by whiplash
- Acute neck injury
- Mild cervical arthrosis

There are no known disorders where the brace cannot or should not be worn.

Avoid applying the brace too lightly in the beginning.


Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING

The neck brace is available in 2 circumferences and 2 heights. Hold the head straight and measure the circumference of the neck from the back over the Adam's apple. The table on the packaging shows the correct brace size for your neck circumference. The required height is determined during fitting. The head is properly supported when it is in a neutral position.

- Hold the brace in both hands in front of you, so that the inside is visible. The point at the centre of the brace should be pointing down.
- Then close the hook and loop fastener at the back of the neck.

WASHING INSTRUCTIONS

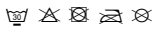Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30° C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

To extend the brace's service life, it should be dried after use, if necessary, before being stored.

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist. Please be aware that an ill-fitting brace does not function properly.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before use, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. Save the product information, to be able to reread the fitting instructions later.
- Consult a specialist if the product requires individual adaptation.
- Check your Push brace each time before using for signs of wear or ageing of parts or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.

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