PUSH CARE WRIST BRACE

INTRODUCTION

Push care products provide solutions when your freedom of movement is restricted by an injury or disorder. Approved by physicians and scientists.

Your new wrist brace is a Push care product, characterised by a high level of effectiveness, comfort and quality. The Push care wrist brace offers pain relief and a sense of greater stability by means of adjustable pressure.

Push care products are innovative and are manufactured for you with great care.

This ensures maximum security and carefree use.

Read the product information carefully before you use the Push care wrist brace.

If you have any questions, please consult your reseller or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

Your Push care wrist brace provides compression around the wrist, improving the sense of stability of the joint. This compression can be adjusted as needed. The wrist joint is not immobilised, leaving you sufficient freedom of movement to carry out activities. The palm of the hand is left free, to allow a proper grip.

So-called comfort zones of padded material have been created around vulnerable areas of the wrist. A proper anatomic fit contribute to the brace remaining in position during movements.

The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push care wrist brace is easy to apply and can be washed without any problem.

INDICATIONS

- Slight strain complaints in the wrist
- Slight distortion
- Slight residual instability
- Mild arthritis
- Slight rheumatoid complaints

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia.

Avoid applying the brace too lightly in the beginning.

Pain is always a sign of warning. If the pain persists or complaints deteriorate, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING


The wrist brace is available in 4 sizes. Measure the circumference of the wrist as shown in the illustration on the packaging. The table on the packaging shows the correct brace size for your wrist.

A The correct fitting sequence is determined by the increasing molecule symbols on the wrist brace. Slide the base around the hand until it reaches the thumb. Close the first fastening around the wrist.

B Wrap the long strap around the wrist to apply the required compression to the wrist.

C Close the hook and loop fastener.

WASHING INSTRUCTIONS

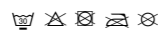Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30°C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

To extend the brace's service life, it should be dried after use, if necessary, before being stored.

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before use, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. Save the product information, to be able to reread the fitting instructions later.
- Consult a specialist if the product requires individual adaptation.
- Check your Push brace each time before using for signs of wear or ageing of parts or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.

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