PUSH MED ELBOW BRACE

INTRODUCTION

Push med products provide effective solutions for the treatment and prevention of injuries of the locomotor apparatus. Approved by physicians and scientists.

Your new elbow brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med elbow brace restricts stretching and provides pain relief. This increases physical strength. Mechanical support and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push med elbow brace. If you have any questions, please consult your reseller or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

Your Push med elbow brace features continuously adjustable extension restriction for the elbow joint. This prevents overstretching. The required degree of restriction can be set easily with one hand. Two stiffeners provide support and direction to the flexion and extension movements. So-called comfort zones of padded material have been created around vulnerable areas of the elbow. A proper anatomic fit and the material and the silicone application contribute to the brace remaining in place during movement.

The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push med elbow brace can be washed normally.

INDICATIONS

- Complaints after hyperextension trauma
- Arthrosis
- Bursitis olecrani
- Olecranon bursitis
- Impingement

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly in the beginning.


Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING

The elbow brace is available in 5 sizes. Straighten the elbow. Measure the circumference of the elbow, as shown in the illustration on the packaging. The table on the packaging shows the correct brace size for your elbow.

- Release all hook and loop fasteners. Hold the inside of your hand towards you while your elbow is bent slightly. Use the stiffener ends to slip the brace on your arm Alternate between the left and right stiffener ends, to ensure that the brace fits symmetrically on the arm. The brace is properly located when you feel the protruding part of the elbow in the opening. Lift your arm to expose the zip and close the zip.
- Use the non-elastic straps to set the required stretching restriction. The stretching movement of the elbow is restricted more as the hook and loop fastener is attached further towards the hand.
- Close the non-elastic strap at the back of the upper arm.

WASHING INSTRUCTIONS

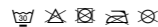Before washing, close the hook and loop material to protect the brace and other laundry.

The brace can be washed at temperatures up to 30°C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

To extend the brace's service life, it should be dried after use, if necessary, before being stored.

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before purchasing, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to be able to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace each time before use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- Within the framework of EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the competent authority of the Member State in which the user is established.

Push® is a registered trademark of Nea International bv

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