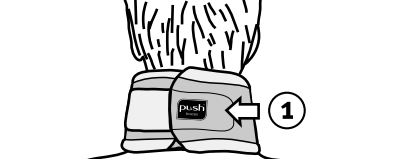
A.

2.

B.

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PUSH MED NECK BRACE

INTRODUCTION

Push med products provide effective solutions in the case of a disorder or reduced functioning of the locomotor apparatus. Approved by physicians and scientists.

Your new neck brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med neck brace provides pain relief by stabilising the cervical vertebrae. Mechanical support and good anatomic fit constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and enhanced functionality. To promote stability and use.

Read the product information carefully before you use the Push med neck brace. If you have any questions, please consult your reseller or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

Your Push med neck brace provides support to the vertebral column by its anatomic shape and the use of semi-rigid foam. The neck brace contains a detachable element for additional support. This element rests on the breastbone and the jaws, for highly effective support. The brace can also be used without the reinforcement element if lighter support is desired. This may be the case when sleeping or as the treatment progresses. A recess is left for the Adam’s apple to enhance comfort. The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push med neck brace is easy to apply and can be washed without any problem.

INDICATIONS

- Follow-up treatment of whiplash
- Acute neck injuries
- Posturgical treatment
- Cervical arthrosis
- Cervical arthrosis with a pseudoradicular complaints
- Tension headache

There are no known disorders where the brace cannot or should not be worn. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING

The neck brace is available in 2 circumferences and 2 heights. Hold the head straight and measure the circumference of the neck from the back over the Adam’s apple. The table on the packaging shows the correct brace size for your neck circumference. The required height is determined during fitting. The head is properly supported when it is in a neutral position.

- Hold the brace in both hands in front of you, so that the inside is visible. The point at the centre of the brace should be pointing down.
- Then close the hook and loop fastener at the back of the neck.

Removing and fitting the reinforcement element:

Place the brace with the inside on a solid surface, with the dot in the centre of the brace pointing towards you. There is a zip on the right-hand side.

Open the zip to remove the reinforcement element. Locate the aperture in the reinforcement element and pull the element out of the brace, while holding the brace tightly. To insert the reinforcement element, perform the actions in reverse order.

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