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### PUSH MED PATELLA BRACE

### INTRODUCTION

Push med products provide effective solutions in the case of a disorder or reduced functioning of the locomotor apparatus. Approved by physicians and scientists.

Your new patella brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med patella brace improves functioning and provides pain relief. This increases physical strength. Mechanical support, adjustable pressure and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push med patella brace. If you have any questions, please consult your reseller or send an e-mail to [info@push.eu](mailto:info@push.eu).

### PRODUCT CHARACTERISTICS

The Push med patella brace applies local pressure on the knee tendon and supports the patella. This relieves the insertion of the thigh muscle and reduces the tractive force on the lower leg.

The pad consists of a skin-compatible and elastic polymer. The synthetic bracing element underneath the pad ensures an even distribution of pressure.

A proper anatomic fit and the material of the pad contribute to the brace remaining in place during walking. The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry.

The Push med patella brace is easy to apply and can be washed without any problem.

### INDICATIONS

- Tendinitis of the patella tendon
- Osgood-Schlatter disease
- Patelofemoral pain syndrome
- Subluxation/hypermobility proximal tibiofubular joint

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

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