## PUSH ORTHO ANKLE BRACE AEQUI

### INTRODUCTION

Push Ortho products provide effective solutions in the case of a disorder or reduced functioning of the locomotor apparatus. Approved by physicians and scientists.

Your new ankle brace is a Push ortho product, characterised by a high level of effectiveness, comfort and quality. The Push Aequi increases ankle stability and helps to control pain. This increases physical strength. Mechanical support, adjustable pressure and good position retention constitute the basis of its performance.

Push ortho products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push Aequi. If you have any questions, please consult your reseller or send an e-mail to [info@push.eu](mailto:info@push.eu).

### PRODUCT CHARACTERISTICS

The Push Aequi is an effective ankle brace, which strikes a balance between efficient mechanical support of an unstable ankle and good wearer comfort. The rigid support is situated exclusively on the medial (inner) side of the ankle and has a cavity for the malleolus. The lateral side is made from comfortable foam, which easily adapts to the individual anatomy. The bracing material is constructed in such a way that it is very rigid in the centre, becoming softer towards the edges to ensure a good fit and wearer comfort. The brace is fastened by means of a diagonal strap. The elastic straps apply pressure around the ankle. The Push Aequi can be used in the functional treatment of ankle injuries and as a long term support in cases of chronic ankle instability. The inversion/eversion movement (inward / outward movement of the foot) is restricted while plantar/dorsal flexion (bending and stretching) is allowed, for adequate freedom. This means that no inhibition is felt during normal walking and running. The numbers on the straps make fitting the brace simply itself. The Push Aequi is worn over the sock, and optimal stabilisation can only be achieved if it is worn with a closed (lace-up) shoe.

### INDICATIONS

- Moderate to serious instability
- Lateral ankle ligament injury
- Follow-up treatment of conservative or operated ankle fractures
- Moderate to serious arthrosis
- Secondary prevention of ankle ligament injury

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too lightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

## ORTHO

ANKLE BRACE AEQUI

KNÖCHELORTHESE AEQUI

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ENKELORTHESE AEQUI

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TOBILLERA AEQUI

T

## PRODUCT INFORMATION

### WASHING INSTRUCTIONS

The brace can be washed by hand at temperatures up to 30°C. Do not use any bleaching agents.

Hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

To extend the brace's service life, it should be dried after use, if necessary, before being stored.

### IMPORTANT

The efficient use of the Push Aequi ankle brace depends on the correct diagnosis of the reasons for pain or instability. Always check first with your doctor, a physiotherapist or another specialist before using a brace.

If this brace is to function effectively it is important:

- to follow the washing instructions
- that the elastic material can still be stretched
- the hook and loop material fastens efficiently
- the brace is not severely damaged (no tears, loose seams)
- the brace is not subjected to temperatures higher than 70°C.
- please be aware that an ill-fitting brace does not function properly.

Push® is a registered trademark of Nea International bv

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