PUSH® ORTHO ANKLE FOOT ORTHOSIS AFO
INTRODUCTION
PUSH® ortho products provide effective solutions in the case of a disorder or reduced functioning of the ankle...


PRODUCT CHARACTERISTICS
The Push AFO provides effective support in the case of foot drop, ensuring minimal disturbance of a person's natural gait...

INDICATIONS
Loss of control of the ankle dorsiflexors during the swing phase and during loading response (heelstrike / foot flat) is a result of failure of the tibialis anterior...

CONTRAINDICATIONS
Loss of lack of control of the calf muscles, moderate to severe spasticity, extreme pes equinus, foot deformity.

FITTING
Open all straps. Insert your foot underneath the crossed straps (A). Place your heel securely at the back of the heel area (B).

INSTRUCTIONS FOR INDIVIDUAL FITTING BY A PROFESSIONAL SPECIALIST
Determining size
The Push AFO is available in three sizes in a left and right execution. Place the foot at a 90-degree angle to the lower leg...

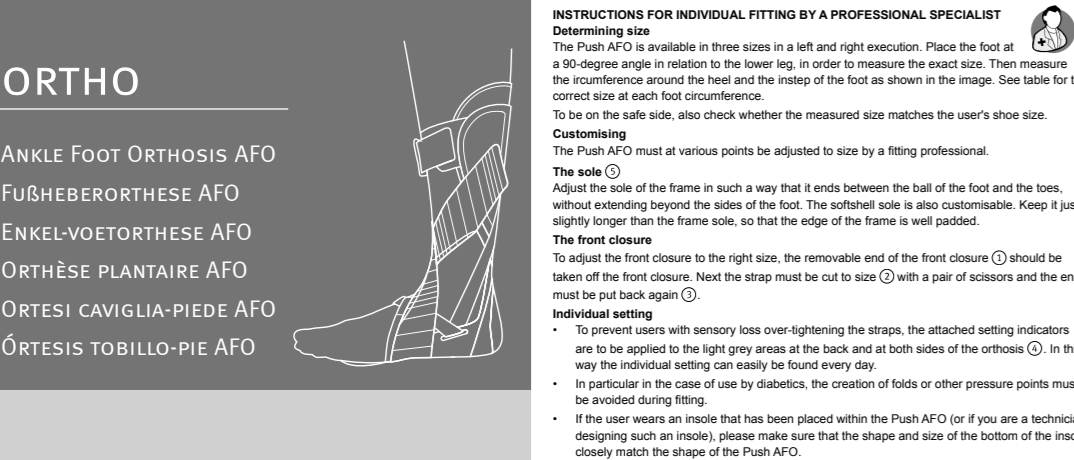PRODUCT INFORMATION

WASHING INSTRUCTIONS
Before washing, close the hook and loop fasteners to protect the ankle and your laundry. The brace can be washed at temperatures up to 30°C on a delicate fabrics cycle or by hand.

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