PUSH ORTHO THUMB BRACE CMC

INTRODUCTION
Push ortho products offer effective solutions for complaints or reduced functioning of the locomotor apparatus. The products have been approved by medical specialists and scientists.

Your new thumb brace is a Push ortho product, characterised by a high level of effectiveness, comfort and quality. The Push thumb brace CMC provides improved stability and pain relief. It increases the ability to bear physical stress. Its optimal fit provides maximum mechanical support and freedom of movement.

Push ortho products are innovative and are carefully manufactured for you. This guarantees maximum security and greater freedom of movement.

Read the product information carefully before you use this product and follow the fitting instructions described below. If you have any questions, please consult your reseller or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS
Osteoarthritis of the carpometacarpal joint of the thumb (CMC-1) causes functional instability of the joint at the base of the thumb as a result of a weakening of the ligaments.

The Push CMC stabilises the basal joint of the thumb (CMC-1) and places the thumb's metacarpal (metacarpal 1) in a functional position.

This creates a functional thumb arc, which remains stable while the hand is used. It relieves the pain when the thumb is under stress. This method of stabilisation leaves the MCP-1 joint and the wrist free, to allow maximum functioning of the hand. For optimal performance, the aluminium core needs to fit closely to the thumb. The product is supplied in the form of a ready-to-use aluminium brace. The brace is formed by reshaping the aluminium core. The selected materials and rounded edges maximise comfort. The simple closing system enables you to fit the brace yourself without any effort.

The Push CMC is a slim and compact product. The brace can be used in the water. After use, simply let it dry before refitting it. As hygiene is important around the hand, the synthetic material is antibacterial and the brace can be machine washed at 40 °C.

INDICATIONS

- CMC-1 osteoarthritis
- Postsurgical stabilisation of the CMC-1 joint
- Instability of the CMC-1 joint

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

FITTING
The instructions for the prescriber/provider can be found under items ①, ②, ③ and ④ below and the illustrations opposite.

The fitting instruction for the user can be found under items A and B below and the illustrations opposite.

- ① Before the initial fitting of the orthosis, gently spread the metal insert to make it slightly larger.
- ② With the orthosis in place, ask the patient to gently touch the tip of the thumb to the tip of the index finger, making an 'O' shape. Be sure the thenar (thumb) muscles are relaxed and NOT actively pinching.
- ③ Looking directly into the palm, be sure the thumb is positioned so the MP and IP joints are slightly flexed and there is slight angle between the 1st and 2nd metacarpal shafts as shown here by the dotted lines.
- ④ Maintain the thumb in the 'O' posture with the thenar muscles relaxed, and firmly squeeze the metal insert to snugly fit the contour and size of the thenar eminence.

CARE AND MAINTENANCE

Before washing, close the hook and loop material to protect the brace and other laundry.

The brace can be washed at temperatures up to 40 °C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used; afterwards, dry the brace naturally. (Do not dry it on a radiator or in a tumble drier).

To extend the brace's service life, it should be dried after use, if necessary, before being stored.

IMPORTANT
Correct use of your brace requires a detailed diagnosis of the cause of your pain, complaints or instability. Always consult your GP, a physiotherapist or a specialist before applying the brace.

For the brace to function properly, the following requirements must be met:

- The brace fits properly. If necessary, an acknowledged expert should adapt the brace to fit.
- The washing instructions must be followed
- The hook and loop material fastens efficiently
- There is no damage (tears, disintegrated components)

Consult the material composition below in this information leaflet.

Push® is a registered trademark of Nea International bv

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