PUSH ORTHO THUMB BRACE CMC

INTRODUCTION

Push ortho products offer effective solutions for complaints or reduced functioning of the locomotor apparatus.

Your new thumb brace is a Push ortho product, characterised by a high level of effectiveness, comfort and quality. The Push thumb brace CMC provides improved stability and pain relief. It increases the ability to bear physical stress. Its optimal fit provides maximum mechanical support.

Push ortho products are innovative and are carefully manufactured for you. This guarantees maximum security and greater freedom of movement. To promote satisfaction with use.

Read the product information carefully before you use this product and follow the fitting instructions described below. If you have any questions, please consult your retailer or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

Osteoarthritis of the basal joint of the thumb (CMC-1) causes functional instability as a result of a weakening of the ligaments.

The Push CMC stabilises the basal joint of the thumb (CMC-1) and places the thumb's metacarpal (metacarpal 1) in a functional position.

This creates a functional thumb arc, which remains stable while the hand is used. It relieves the pain when the thumb is under stress. This method of stabilisation leaves the MCP-1 joint and the wrist free, to allow maximum functioning of the hand.

For optimal performance, the aluminium core needs to fit closely to the thumb. The specialist supplying the product can, if necessary, adjust the brace individually by changing the aluminium core. The selected materials and rounded edges maximise comfort. The simple closing system enables you to fit the brace yourself without any effort.

The Push CMC is a slim and compact product. The brace can be used in the water. After use, simply let it dry before refitting it. As hygiene is important around the hand, the synthetic material is antibacterial and the brace can be machine washed at 40°C.

INDICATIONS

- CMC-1 osteoarthritis
- Postsurgical stabilisation of the CMC-1 joint
- Instability of the CMC-1 joint

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly.

Pain is always a warning sign. If the pain persists or symptoms worsen, it is advisable to consult your doctor or therapist.

FITTING

The instructions for the prescriber/provider can be found under items ①, ②, ③ and ④ below and the illustrations opposite.

The fitting instruction for the user can be found under items A and B below and the illustrations opposite.

① Before the initial fitting of the orthosis, gently spread the metal insert to make it slightly larger.

A Place the orthosis on the hand/thumb and push it as far down as possible toward the base of the thumb.

B First close the lower strap nearest to the wrist. Then close the upper strap.

② With the orthosis in place, ask the patient to gently touch the tip of the thumb to the tip of the index finger, making an "O" shape. Be sure the thenar muscles are relaxed and NOT actively pinching.

③ Looking directly into the palm, be sure the thumb is positioned so the MP and IP joints are slightly flexed and there is slight angle between the 1st and 2nd metacarpal phalanges, as shown here by the dotted lines.

④ Maintain the thumb in the "O" shape with the thenar muscles relaxed, and firmly squeeze the metal insert to snugly fit the contour and size of the thenar eminence.

CARE AND MAINTENANCE

The product should be washed before being used for the first time. To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 40°C, on a delicate fabric cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble dryer).

IMPORTANT

• Correct use of your brace requires a detailed diagnosis of the cause of your pain complaints or instability. Always consult your GP, a physiotherapist or a specialist before applying the brace.

• Push braces are designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.

• Optimal functioning can only be achieved by selecting the right size. Try on the brace before purchasing, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to consult it for fitting instructions whenever needed.

• Consult a brace specialist if the product requires individual adaptation. Check your Push brace before each use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.

• In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the appropriate regulatory body in the user's country of residence.

Consult the material composition below in this information leaflet.

Push® is a registered trademark of Nea International bv

For details of the distributor in your country, refer to www.push.eu/distributors

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