METAGRIP® THUMB CMC ORTHOSIS

INTRODUCTION

The MetaGrip® is an innovative and carefully manufactured product, providing maximum stability of the thumb CMC joint while allowing freedom of movement of other joints.

Read this information carefully before applying the MetaGrip®. If you have any questions consult your reseller or send an e-mail to info@push.eu.

INDICATIONS/PRECAUTIONS

The MetaGrip® is recommended for use with:

- Thumb CMC joint osteoarthritis
- Thumb CMC joint instability
- Stabilization of thumb CMC joint following arthroplasty

For the orthosis to be effective, it must be fitted properly and applied correctly. Avoid applying the MetaGrip® too tightly and discontinue use of the orthosis and report to a medical professional if there is persistent pain or pressure areas.

FITTING THE METAGRIP®

Wash before first use.

1. Measure the circumference of the hand proximal to the metacarpal heads to choose the correct size:

	Size 0	5 7/8 - 6 7/8 inches
	Size 1	6 7/8 - 7 3/4 inches
	Size 2	7 3/4 - 8 7/8 inches.
	Size 3	8 7/8 - 10 5/8 inches

2. Before the initial fitting of the orthosis, gently spread the metal insert to make it slightly larger (Figure ①).
3. Place the orthosis on the hand/thumb and push it as far down as possible toward the base of the thumb (Figure ②, ③, ④). First close the lower strap nearest to the wrist. Then close the upper strap.
4. With the orthosis in place, ask the patient to gently touch the tip of the thumb to the tip of the index finger, making an 'O' shape. Be sure the thenar (thumb) muscles are relaxed and NOT actively pinching (Figure ⑤).

5. Looking directly into the palm, be sure the thumb is positioned so the MP and IP joints are slightly flexed and there is slight angle between the 1st and 2nd metacarpal shafts as shown here by the dotted lines (Figure ⑥).

6. Keep the thumb in the 'O' posture with the thenar muscles relaxed, and firmly squeeze the metal insert to snugly fit the contour and size of the thenar eminence (Figure ⑦).

NOTE: if there is a prominent bony contour at the dorsal base of the thumb, it may be desirable to use a pair of needlenose pliers to accurately contour the metal insert around the bony prominence.

CARE AND MAINTENANCE

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

The orthosis can be washed at temperatures up to 104 °F on a delicate fabrics cycle or by hand. DO NOT use bleaching agents.

Before washing, close the straps to protect other laundry. A gentle spin cycle may be used followed by air drying.







DO NOT dry the orthosis on a radiator or in a clothes dryer or wash/dry in a dishwasher.

GUARANTEE

The MetaGrip® orthosis is guaranteed against defects in materials and workmanship for a period of 90 days from purchase. Modifications to the orthosis void this warranty.

IMPORTANT

- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Check your Push brace each time before use for signs of wear or aging of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the appropriate regulatory body in the user's country of residence.

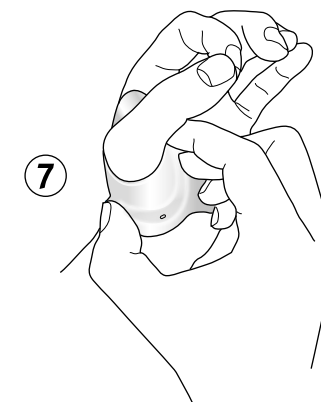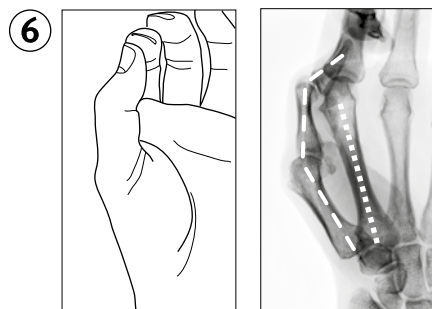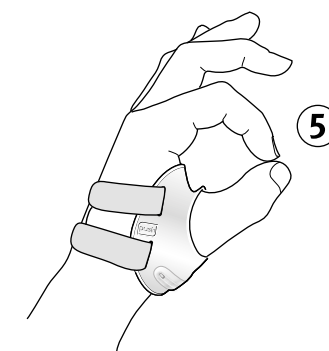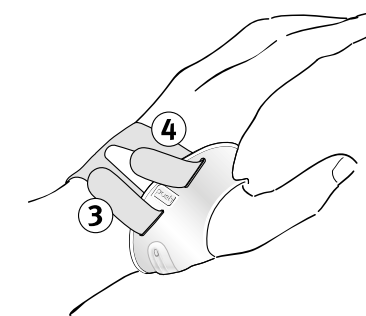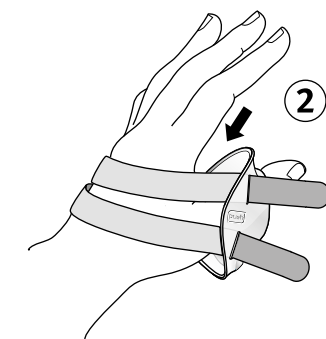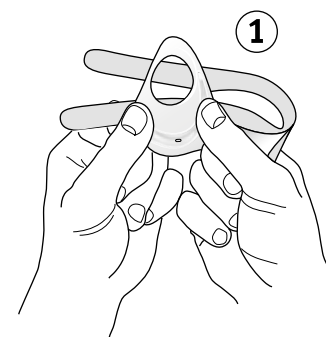PRODUCT INFORMATION

ORTHO

METAGRIP® THUMB CMC ORTHOSIS
METAGRIP® ORTESIS PARA PULGAR CMC

MODEL

3.10.2

Nea International bv
Europalaan 31
6199 AB Maastricht-Airport
The Netherlands

www.push.eu

Designed in the Netherlands
Made in the EU

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© Nea